| SPECTRA Disposable Radio Frequency (RF) Needle | |
|--|---------------------|
| English | Instruction for Use |
| Indications for Use: The SPECTRA Disposable Radio Frequency (RF) Needles are indicated for use in radio frequency heat lesion procedures for relief of pain. | |
| Instructions for Use: | |
| <ol style="list-style-type: none"> 1 Remove the needle guard. 2 Read and follow all instructions for use when placing a Spectra Disposable Radio Frequency (RF) Needle for the subsequent medical procedure. 3 The physician will place the cannula under fluoroscopic (C-Arm) guidance. Once the cannula has been positioned the stylet may be removed. 4 The thermocouple (TC) or thermistor (TM) electrode can be inserted into the needle. 5 Follow the Manufacturer's instructions for Use for the TC electrode and RF generator. 6 When the procedure is complete, remove the TC or TM and dispose of per Manufacturer's instructions. 7 Then carefully remove the Needle and place it in a Sharps Container. | |

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0173-001-03_SMD-IFU-0009 Rev 004

| SPECTRA | |
|--|--|
| | |
| Warnings: | Cautions: |
| <ul style="list-style-type: none"> • This product is for single use, DO NOT REUSE - DO NOT RSTERILIZE. Reuse and/or repackaging may create a risk of patient or user infection, compromise the structure integrity and/or essential material and design characteristics of the device, which may lead to device failure, and/or injury, illness or death of the patient. • Do not use if package is opened or damaged. • Do not alter this device or Straighten Bent Needle – Discard if Bent. • DO NOT AUTOCLAVE. • This device may cause interference in proximity of a magnetic resonance imaging machine. • Handle and discard in accordance with accepted medical practice and applicable regulations. After use this product is a potential biohazard. • Re-sheathing of needles is hazardous. | <ul style="list-style-type: none"> • Carefully read and follow all relevant hospital/medical institutional guidelines when placing a Radio Frequency (RF) Needle for the subsequent medical procedure. • Care and adherence to universal precautions must be taken to avoid accidental needle stick injuries. • Only qualified healthcare practitioners with proper training should insert, manipulate and remove these devices. • Follow standard infection control precautions as specified by the CDC (USA) or local equivalent. • Federal (U.S.A.) Law restricts this device to sale by or on the order of a physician. |
| NOTE: Refer to the IFU from the manufacturer of the thermocouple (TC) or thermistor (TM) and the RF Generator with which the Spectra Disposable Radio-Frequency Needle is being used for additional WARNINGS and CAUTIONS. | |

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0173-001-03_SMD-IFU-0009 Rev 004

| SPECTRA Disposable Radio Frequency (RF) Needle | |
|--|---|
| English | ADVERSE EVENTS |
| 1 | Potential complications from procedure with which this Device is used include, but are not limited to: Mild Discomfort; Temporary Pain; Paresthesia; Local Pain; Numbness of Back; Dyspareunia; Soreness; Increased intensity of low back pain; Intracranial Hypotension; Bradycardia; Transient Thermal sciatic neuropaxia; Inflammation; Disc prolapse. |
| 2 | Potential device problems related to the RF Needle: Break / Fracture / Crack / defective; Material Separation / peeled / delaminated / Erosion / Fragmentation / discolored / corroded / protrusion / extrusion; Device misassembled during shipping / Detachment of device component; Contamination problem; Leak / Splash. |

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